## DAC-SpectroMed s.r.l.

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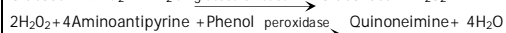
### PRINCIPLE

Glucose in the sample originates, by means of the coupled reactions described below, forms a colored complex that can be measured by spectrophotometry.

### CONTENTS AND COMPOSITION

Reagent	pH 7.5
Phosphate	70 mmol/l
Phenol	15 mmol/l
Glucose oxidase	> 10 U/ml
Peroxidase	> 1 U/ml
4-aminoantipyrine	0.4 mmol/l

Standard Glucose Aqueous primary standard.

*NB Calibration with the provided aqueous standard may cause a matrix related bias. In these cases, it is recommended to calibrate using a serum based standard*

### STORAGE AND STABILITY OF REAGENTS

Store reagents at 2-8°C and use until the expiry date shown on the label. Indications of deterioration: presence of particulate material, turbidity, absorbance of the blank over 0,150 at 500 nm (1 cm cuvette).

### SAMPLES

Whole blood, serum free of hemolysis, plasma, urine. Serum or plasma must be separated from the red cells promptly to prevent glycolysis. The addition of sodium fluoride to the blood sample prevents glycolysis. Heparin, EDTA, oxalate and fluoride may be used as anticoagulants.

Standard, blank and samples of whole blood preparation:

Add 40 µl of blood to 1.5 ml of physiological solution, centrifuge mixture 10 minutes at 3000 rpm. For Standard (40 µl) and Blank (40 µl of distilled water) the same procedure is used.

Sample should be 50-fold diluted if glucose level in urine is too high.

Glucose in serum or plasma is stable for 5 days at 2-8°C.

### REFERENCE VALUES

Serum, plasma<sup>2</sup>: 3,85 – 6,11 mmol/l (70-110 mg/dl)

Blood: 3,38 – 5,55 mmol/l (61-100 mg/dl)

Urine: 0-0,2 %, 0-1,11 mmol/l (0-20 mg/dl)

These ranges are given for orientation only; each laboratory should establish its own reference ranges.

### ADDITIONAL EQUIPMENT

Analyzer, spectrophotometer or photometer able to read at 500±20 nm. Thermostatic water bath at 37°C. Pipette for 10 µl, 1,0 ml.

### PRECAUTION

For in vitro diagnostics only. Handle all patients' samples as potentially dangerous and treat as infectious.

### QUALITY CONTROL

It is recommended to use the Sera N and Sera P to verify the performance of the measurement procedure.

Each laboratory should establish its own internal Quality Control scheme and procedures for corrective action if controls do not recover within the acceptable tolerances.

### REAGENT PREPARATION

Reagent and Standard are provided ready to use.

### PROCEDURE

Assay conditions

Wavelength:	500 (±20) nm
Temperature:	16-25/37 °C
Cuvette:	1 cm light path
Method:	endpoint (increasing)
Read against:	reagent blank

*NB: Volumes of reagent, standard and samples can be proportionally changed according to the cells working volumes of using analyzers.*

1. Bring the Reagent to room temperature (15-25°C).

A. Whole blood

2. Pipette into labeled test tubes:

	Test	Blank
Supernatant (blood, Standard), ml	0.50	-
Supernatant (distilled water), ml	-	0.50
Reagent, ml	1.00	1.00

3. Mix thoroughly and incubate the tubes for 15 minutes at 37°C.

4. Measure the absorbance of tested Sample and Standard relative to Blank. Wave-length is 500(±10) nm.

Coloration is stable for 40 minutes.

B. Serum (plasma), urine

2. Pipette into labeled test tubes:

	Test	Blank
Sample (Standard), µl	10	-
Distilled water, µl	-	10
Reagent, ml	1.00	1.00

3. Mix thoroughly and incubate the tubes for 15 minutes at 37°C.

4. Measure the absorbance of tested Sample and Standard relative to Blank. Wave-length is 500(±10) nm.

Coloration is stable for 40 minutes.

### CALCULATIONS

The glucose concentration in the sample is calculated using the following general formula:

$$\frac{A_{\text{Sample}}}{A_{\text{Standard}}} \times C_{\text{Standard}} = C_{\text{Sample}}$$

### METROLOGICAL CHARACTERISTICS

Detection limit: 0.23 mg/dl = 0.0126 mmol/l

Linearity limit: 500 mg/dl = 27.5 mmol/l.

For higher values dilute sample 1/4 with distilled water and repeat measurement.

Repeatability (within run):

Mean Concentration	CV*	n*
88 mg/dl = 4,84 mmol/l	1,2%	20
326 mg/dl = 17,93 mmol/l	0,9%	20

Reproducibility (run to run):

Mean Concentration	CV*	n*
88 mg/dl = 4,84 mmol/l	2,7%	25
326 mg/dl = 17,93 mmol/l	1,9%	25

\* CV - coefficient of variation n – number of determinations

Sensitivity: 4 mA dl/mg = 0,22 mA l/mmol.

Interferences: Hemoglobin (> 5 g/l), lipemia (triglycerides > 1.25 g/l) and bilirubin (10 mg/dl) may interfere. Other substances and drugs may interfere<sup>4</sup>. Results may vary if a different instrument or manual procedures are used.

### DIAGNOSTIC CHARACTERISTICS

Glucose is the major source of energy in the body. Insulin, produced by islet cells in the pancreas, facilitates glucose entry into the tissue cells. A deficiency of insulin or a decrease of its effectiveness increases blood glucose. Elevated serum or plasma glucose concentration is found in diabetes mellitus (insulin dependent, non-insulin dependent) and in other conditions and syndromes<sup>2,3</sup>. Hypoglycemia can occur in response to fasting, or it may be due to drugs, poisons, inborn errors of metabolism or previous gastrectomy<sup>2,5</sup>.

Clinical diagnosis should not be made on the findings of a single test result, but should integrate both clinical and laboratory data.

### BIBLIOGRAPHY

- Trinder P. Determination of glucose in blood using glucose oxidase with an alternative oxygen acceptor. Ann Clin Biochem 1969; 6: 24-27.
- Tietz NW. Clinical guide to laboratory tests, 2nd ed. Saunders Co, 1991.
- National Diabetes Data Group: Classification and diagnosis of diabetes mellitus and other categories of glucose intolerance. Diabetes 1979; 28:1039-1057.
- Young DS. Effects of drugs on clinical laboratory tests, 4th ed. AACC Press, 1995.
- Friedman and Young. Effects of disease on clinical laboratory tests, 3th ed. AACC Press, 1997